









# Ohio Medical's Medical Gas Blenders



## DESCRIPTIONS AND APPLICATIONS

Ohio Medical's  $^{\text{TM}}$  Medical Air/ $O_2$  blenders combine compressed Medical Air and Oxygen to deliver blended pressurized gas at a precise oxygen concentration (FiO<sub>2</sub>) determined by the user.

Some models are equipped with a unique gas bleed ON/OFF switch to increase accuracy when needed, conserve gas and reduce worries.

We offer models with multiple outlet ports that deliver the same highly accurate selected FiO<sub>2</sub>. Most models can be customized to include a flowmeter attachment with a variety of flow rates available. The blenders contain an audible alarm which warns the user of inaccurate flow to the patient.

These blenders are suitable for respiratory applications including routine therapy, ventilator gas supply, sophisticated life-support and critically-limited NICU procedures.

#### Did You Know...

All Medical Air and Oxygen blenders require a bleed for accuracy. The bleed on a blender is activated any time something is connected to the right port.

This means that gas is constantly being bled into the air even when the blender is not in use.

# A Unique Product In An Extensive New Line

The NEO<sub>2</sub> Blend offers an impressive and economical feature - the unique Bleed Switch. Unlike most other blenders, the NEO<sub>2</sub> Blend allows you to turn off the low flow gas bleed without disconnecting anything from the blender. The blender is always ready to use when you need it.

This NEO<sub>2</sub> Blend was specifically designed for use in the Neonatal Intensive Care Unit; it comes with two assembled flowmeters. The built-in left side Flowmeter flows from 1-15 lpm, while the right side Flowmeter flows from 0.1-1 lpm. These built-in Flowmeters allows for a smaller footprint in the NICU where space is very limited.

The NEO<sub>2</sub> Blend conserves gas when the blender is not in use. The right flowmeter is part of the switch. Rotate the flowmeter forward 45° to turn off the bleed when not in use, or if not needed during use. Rotate it back to vertical to turn the bleed back on. The Bleed ON/OFF switch is both easy to do and clearly visible.

Our switch saves you time, aggravation, gas and money.



# Ohio Medical makes a blender for your every need!

# **LOW** FLOW BLENDERS

Perfect for NICU and Newborn Nursery or for a flow of 30 lpm or less. Precise accuracy at even the lowest flow range.









These blenders allow the ability to choose the bleed flow for increased accuracy or to turn it off easily for significant gas savings

# **HIGH/LOW** FLOW BLENDERS

Versatile dual range blender. Ideal for equipment needing flows from 2 -120 lpm, yet also capable of accuracy at even a low flow range.









### JUST THE FACTS

- Configure the blender exactly how you need it by choosing exactly what flowmeter type suits your needs
- 3 models offer gas savings ON/OFF bleed feature
- · Available in very low to high flow capability
- Suited to specialty and general needs
- Mounting solutions for all situations
- All blenders are equipped with an audible alarm

#### MOUNTING BRACKETS

#### **Dovetail Rail Version**



6600-0031-900

#### **Wall Version**



6600-0044-500

#### **Pole Mount Version**



6700-0645-800

Other configurations available; please speak to your sales representative for details.

## HOSE KITS





DISS: Ohmeda: Chemetron: Puritan Bennett:



6700-4003-815 6700-4001-815 6700-4002-815 6700-4004-815

#### **10ft Hose Kit**

10' Air and 10' Oxygen supply hoses



DISS: 6700-4008-815
Ohmeda: 6700-4005-815
Chemetron: 6700-4007-815
Puritan Bennett: 6700-4009-815

### O R D E R I N G M A T R I X

DESCRIPTION	PART NUMBER
NEO <sub>2</sub> Blend with assembled 0-15 lpm and 0-1 lpm Flowmeters. Also equipped with a unique on/off bleed switch	6750-0018-907
Low Flow Blender with on/off Bleed Knob (One Left Port)	
Plain/No Flowmeters	6750-0019-907
Attached 0-1 lpm Flowmeter (left)	6750-0019-908
Attached 0-3.5 lpm Flowmeter (left)	6750-0019-909
Attached 0-15 lpm Flowmeter (left)	6750-0019-910
Low Flow Blender (Left and Right Port)	
Plain/No Flowmeters	6750-0020-907
Empty (left)/ Attached 0-15 lpm (right) Flowmeter	6750-0020-910
Attached 0-15 lpm (left) / 0-1 lpm (right) Flowmeters	6750-0020-918
Attached 0-15 lpm (left)/ 0-3.5 lpm (right) Flowmeters	6750-0020-919
High/Low Blender (Right and Bottom Port)	
Plain/No Flowmeters	6750-0024-907
Attached 0-15 lpm Flowmeter	6750-0024-910
High/Low Blender with 3 Ports	
Plain/No Flowmeters	6750-0025-907
Empty (left)/ Attached 0-15 lpm (right) Flowmeter	6750-0025-910
High/Low Blender with an assembled 0-15 lpm Flowmeter and unique on/off bleed switch	6750-0026-910

Specifications are nominal, subject to change without notice. International options are also available.

Note: Ohio Medical Air/O<sub>2</sub> Blenders are compatible with Ethylene Oxide Gas (ETO) cold sterilization.

Attached Flowmeters are connected with DISS Handtight fittings.

Other configurations available; please speak to your sales representative for details.

